



D-4764-2017

EyeMax 2

Phototherapy Eye Mask

Our latex-free **EyeMax 2** eye mask helps to provide optimum comfort and protection for patients during phototherapy treatments. The one piece, wrap-around design has two attachment points that can be adjusted independently; this can prevent unwanted movement and allows for a customized fit. The ergonomic eye-protection pad was specially designed to prevent ultraviolet light leakage.

Dräger

Technology for Life

Benefits



Extra Protection

A separate light-blocking material is included inside the eye covering portion of the eye mask to **shield retinas from harmful damage.**



Secure Fit

Three different sizes that are each fully adjustable by **two separate velcro points.**




Gentle on Skin

Latex-free, gentle material was created with the fragility of patients in mind with **no adhesives to irritate skin.**



Lightweight Material

Light is able to pass through the thin material to allow **increased skin exposure.**

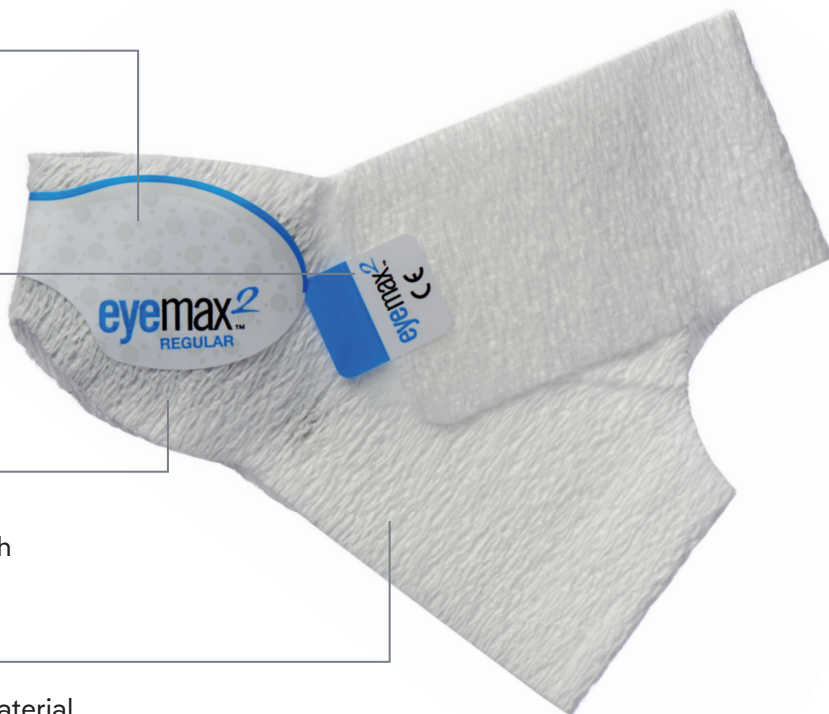

EyeMax 2 Size Guide



Regular (12.6"-14.9")
(Occipital Frontal Circumference)



Preemie (10.4"-12.6")
(Occipital Frontal Circumference)



Micro (7.87"-10.4")
(Occipital Frontal Circumference)

D-4764-2017

Ordering Information

Description	Quantity	Part number
EyeMax 2, Regular (blue)	20 pack	MP03772
EyeMax 2, Premie (orange)	20 pack	MP03771
EyeMax 2, Micro (green)	20 pack	MP03770

How To Use The Eyemax 2

Indications for Use

The EyeMax2 provides neonatal eye protection during ultraviolet (UV) phototherapy treatment of jaundice. The EyeMax2 conforms with EN 170:2002 Clause 5.2 (Scale Number 2-5) for spectral transmittance as appropriate for the indication for use.

Instructions for use

1. Measure Occipital-Frontal Circumference (OFC) of the patient.
2. Select the appropriate EyeMax2 size based on the OFC. See chart for appropriate sizing.
3. Disconnect the Velcro.
4. Position the eye pads over the eyes.
5. Pass the lower strap under the small of the neck and attach velcro.
6. Secure the Velcro of both straps and position the EyeMax2 as shown.
7. Make sure the strap completely covers the ears.
8. If the eye patch rides up away from the nose, tighten the lower strap.
9. If the eye patch rides down towards the nose and mouth, repeat application process beginning with step 1.

Symbol Guide

The following symbols and safety labels are found on the EyeMax2:

 Medical Device	 RX Only
--	---

Precautions



CAUTION

Improper EyeMax2 sizing may lead to patient airway obstruction or inadequate eye protection. The Eyemax2 should only be used while the patient is under the continuous, direct supervision of healthcare professionals.



DO NOT tug on the EyeMax2 to reposition it. A premature infant's skin is very delicate and can easily rupture. Avoid obstructing the nose. Check regularly to ensure the EyeMax2 is not blocking the patient's airway. The EyeMax2 is not a bandage. Cover any existing skin abrasions with appropriate bandages before use.



DO NOT over tighten - causing excessive pressure on the eyes. Make sure the strap is covering the ear, not hooked over the ear, and that the ear is not folded over. Replace the EyeMax2 with a new one if it becomes over-stretched. Reuse of this device could cause cross-contamination or product malfunctions.

Regular Size: occipital-frontal circumference 33-38 cm.

Premie Size: occipital-frontal circumference 26-32 cm.

Micro Size: occipital-frontal circumference 20-25 cm.



Single Patient Use Only
Non-Sterile, Latex-Free

Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released.
Go to www.draeger.com/trademarks to find the current status.

Corporate Headquarters

Drägerwerk AG & Co. KGaA
Moislinger Allee 53-55
23558 Lübeck, Germany

www.draeger.com

**Manufacturer for EyeMax 2
Phototherapy Eye Mask:**

Maxtec
2305 South 1070 West
Salt Lake City, Utah 84119
USA

USA

Draeger, Inc.
3135 Quarry Road
Telford, PA 18969-1042
Tel +1 800 4DRAGER
(+1 800 437 2437)
Fax +1 215 723 5935
info.usa@draeger.com